

# Babybay® Medicare Cleaning Instructions



## Frequency

- ◆ Wipe clean after each patient use
- ◆ Check for visible dirt / contamination and surface damage every day

## Cleaning & Disinfection

- ◆ Remove the mattress, clean both mattress and frame using a manual scrubbing / wiping method



## Recommended Cleaning Agents

- ◆ Babybay® Medicare has been tested by Guest Medical, manufacturers of Chlor Clean<sup>1</sup>
- ◆ Babybay® Medicare can be cleaned using Chlor Clean or other branded chlorine based cleaning agents at the Department of Health recommended guideline strength of 1,000 ppm (0.1%)
- ◆ In instances of body fluid spills or specific deep clean routines, Babybay® Medicare can be cleaned with a chlorine based cleaning agent at a strength of 10,000 ppm (1%).
- ◆ When dry, this may result in a white residue showing on the frame, this is standard and should be wiped off as part of a normal cleaning routine

## Method

- ◆ Wipe mattress both sides and again round zip area using disinfectant wipes, set aside to air dry
- ◆ Clean Babybay Medicare® Frame using wipes to clean all horizontal and vertical bars.
- ◆ Wipe the underside of the horizontal bars and ensure that all surface area of the Babybay comes into contact with cleaning agents
- ◆ Leave Babybay to air dry in a well ventilated area before replacing mattress

## **DO NOT ...**

...use machines, scourers or abrasive implements!

... use methods or materials other than those stipulated in these instructions!

... machine wash, bleach, iron or dry-clean any Babybay Accessories!

<sup>1</sup> Full Chlor Clean Report available on request.